

SAMPLING GUIDE - FRY

Sampling for PCR – first feeders and fry 2 grams

Kidney – V cut

Use a V-cut when sampling kidney from first feeders and 2 grams fry. Place the organ package on the inside of the scalpel package and cut it into two (if enough tissue) pieces of about 2x2x2 mm. Make sure the tissue pieces are covered by RNAlater.



Heart – A cut

Use a A-cut when sampling heart from first feeders and 2 grams fry. Place the organ package on the inside of the scalpel package and cut it into two (if enough tissue) pieces of about 2x2x2 mm. Make sure the tissue pieces are covered by RNAlater.

